

Each Steraffirm Sterilizer Control Tube consists of a 45mm x 5mm borosilicate glass tube sealed at both ends. One end is color coded to facilitate identification of the unboxed indicator. The tube contains approximately 200 µl of heat-sensitive red liquid which changes color from red to green at various times and temperatures that are integrated by the device.



Steraffirm® Control Tubes

for Steam and Dry Heat Sterilization

FEATURES

Color change from red to green after exposure to specified parameters

Permanent color change following exposure

Lot number and expiration on each box

Several exposure time and temperature specific versions are available

Conform to ISO 11140-1, steam Class 4, dry heat Class 6

BENEFITS

Easy to interpret, instant readout after the cycle

The processed Control Tubes may be stored for record purposes

Enhances traceability


Can be used with common dry heat or moist heat cycle parameters

Meets critical standards as a multiparameter indicator

The Control Tubes are calibrated to respond to specific temperature and time parameters during a dry or moist heat sterilization cycle. They have a permanent color change when the proper parameters are achieved at the location of each Control Tube. A color guide in the instruction provides an instant visual check of the results. Since the Control Tubes consist of sealed vials containing heat-sensitive solution, chemicals from the indicator do not normally come into contact with adjacent items during the cycle.

The Control Tubes for steam sterilization can be used for gravity, prevacuum or liquids sterilization with specific time and temperature parameters. The Control Tubes for 121°C /250°F steam can be used to monitor liquid cycles. The Control Tube may be placed in a test bottle of the solution to be sterilized and placed in the slowest-to-heat area of the chamber for routine monitoring of a validated liquid cycle.

The Control Tubes for dry heat are calibrated to some of the more commonly used time and temperature parameters of dry heat sterilization. The Steraffirm Control Tubes are for life sciences applications only. They are not approved for use in healthcare facilities in the United States.

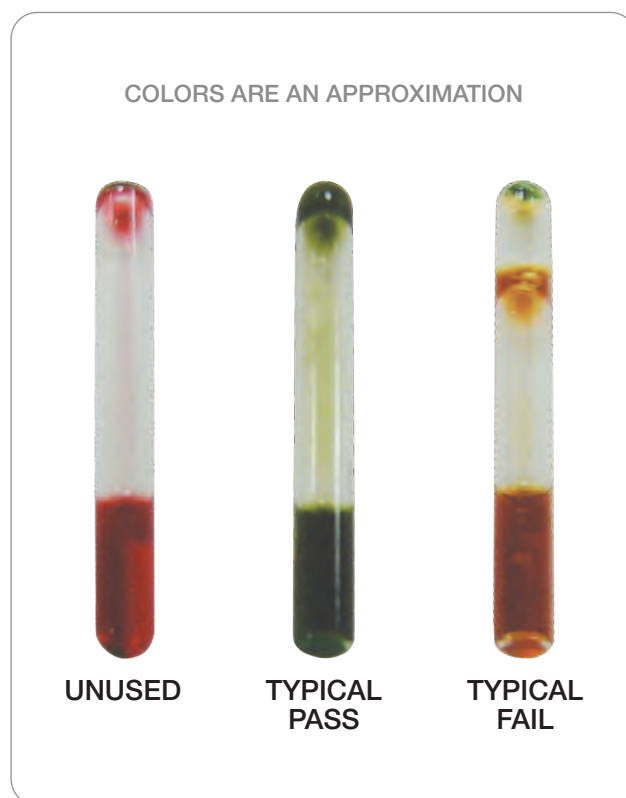

PRODUCT	DESCRIPTION	QUANTITY
PCC024	Steraffirm Control Tubes for Steam 121°C/250°F for 15 minutes	100/box
PCC025	Steraffirm Control Tubes for Steam 121°C/250°F for 12 minutes	100/box
PCC027	Steraffirm Control Tubes for Dry Heat 160°C/320°F for 60 minutes	100/box
PCC028	Steraffirm Control Tubes for Dry Heat 160°C/320°F for 120 minutes	100/box



STERIS CORPORATION
5960 Heisley Road
Mentor, OH 44060, USA
T +1 800 444 9009

www.STERISLifeSciences.com

STERIS Offices Worldwide

Asia Pacific	+65 (0) 68 41 7677
Canada	+1 800 444 9009
Finland	+358 9 25851
France	+33 (0) 2 38 84 85 40
Germany	+49 (0) 221 466120
Italy	+39 (0) 2 21303 424
Latin America	+1 800 444 9009
Spain	+34 (0) 916 585 920
United Kingdom	+44 (0) 1256 840400

STERIS Life Sciences | Science & Solutions for Life

For over 100 years, STERIS has been known as a global leader, trusted partner and solutions provider in the field of sterilization and contamination control. *Today*, STERIS continues building on this heritage by advancing the **Science** of sterilization, cleaning and infection control while offering **Solutions** that meet our Customers' needs and high standards. STERIS is dedicated to helping you enhance the **Life** of your patients and the life of your equipment. From Formulated Chemistries, to Capital Equipment, to Parts and Services, STERIS Life Sciences is **Science & Solutions for Life**.

© 2015 STERIS Corporation
All rights reserved

No part of this publication may be reproduced in any material form without the permission of STERIS Corp.